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EN MASKS, RESPIRATORS FFP2 and FFP3

Product characteristics: Protective masks are light and protect the environment from spreading of harmful substances. Respirators are protective tool against particles, bacteria and viruses.

This is a single use product.

The respirator or respiratory mask filter the air you inhale and capture harmful substances so that they do not enter your respiratory system. Filtering properties are time limited depending on the degree of contamination of the surrounding environment.

Particle filtration efficiency: FFP2 at least 94 %
FFP3 at least 99 %

Application: personal protection, single use

Harmonized standard for respirators: EN149: 2001 + A1: 2009

Quality that meets the respirator standard: EN149: 2001 + A1: 2009

Class of protection: FFP2, FFP3 for respirators

Information about conformity: You can find the EU declaration of conformity on the website www.lamps4sale.co.uk

What it protects us from:

The mask is mainly used to prevent microorganisms from spreading further into the surroundings.

The FFP2 respirator protects the user against solid and liquid harmful dusts, aerosols and fumes. A respirator with filter class FFP2 filters at least 94% of aerosols (air particles) down to a size of 0.6 µm (micrometer).

The FFP3 respirator protects the user against solid and liquid harmful dusts, aerosols and fumes. A respirator with filter class FFP3 filters at least 99% of aerosols (air particles) down to a size of 0.6 µm (micrometer).

Warning: Before use, check if the packaging or mask are not damaged. If the product is damaged, don't use it. Use the respirator or mask as soon as possible after opening the packaging. You shouldn't wear it for more than 12 hours. It cannot be washed. If the user's beard is in contact with the respirator or mask, leaks will occur. Do not use in an environment where the percentage of oxygen is reduced.

Instructions for use: 1. Put the mask on the face so that the nose and mouth are covered. 2. The mask is adjusted to the face by using a nose clip and a pair of elastic ear bands. 3. After putting on the mask, make sure the compressed nose clip follows the shape of the nose and pull the lower end of the mask to your jaw to cover it. 4. For proper use, make sure the mask is covering the entire face from jaw to nose.

Terms of storing: The product needs to be stored in a room with a temperature below 50 °C and a relative humidity of no more than 80%, with good ventilation, no corrosive gases, fire-proof, moisture-proof, rat-proof and insect-proof equipment. The storage period must be in accordance with the validity of the product.

Packaging: Prevent mechanical damage and contamination before use by sealed packaging in sterile and dust-free facilities.

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